

Hy21® Emesis Support and Hygie Hygienic Cover® - Emesis bag

Set-Up **IMPORTANT: When using this device, it is recommended to wear gloves.**

1 Remove the super-absorbent pad from the Emesis Bag and set it aside. 2 Install the Emesis Bag by passing it under the Emesis Support, making sure the Bag is folded over all sides of the support. 3 Fold the super-absorbent pad in 2 then place it at the bottom of the Emesis Bag. The Emesis Support is now ready for use.



Removal after use



Blocks odors from spreading



Reduces cleaning tasks

After patient use, it is suggested to wait until the gelling process has started before proceeding with the removal of the Emesis Support. 4 Remove the Emesis bag from the support. 5 While you remove the Emesis Bag from the bottom of the device, make sure to close the opening. 6 Pull drawstrings (green and white) on one side of the Emesis Bag. Twist the Bag 3 times, make a knot and DISPOSE.

IMPORTANT: Once the Emesis Bag is disposed, remove your gloves and wash your hands.



Product code: HY-SUPP-VOMO-000
Printed in Canada

www.hygie.com

Toll Free : 1 (866) 588-2221

Fax : (450) 444 6222

Canada, USA